




Swedish Hospital Supply AB

P-BAG



Mount the funnel into the receiver on the bag, the funnel is mounted when it "clicks" into the recipient. Make sure that the funnel handle is aligned on the top of the bag as in the picture to the right.

Scan the code below or visit our website for instructional video.

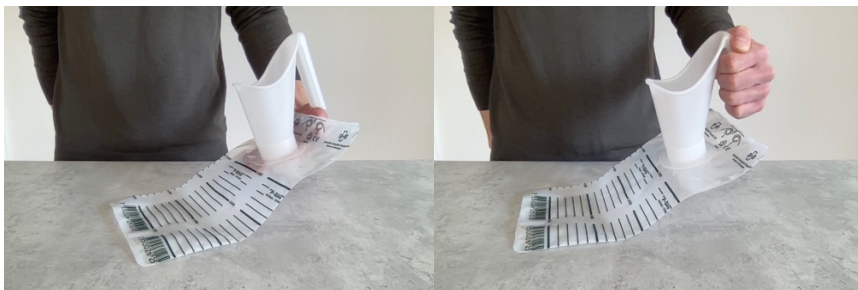

The P-BAG can now be used/delivered to the patient.

Always make sure that the handle of the funnel is pointed towards the top of the bag.
Note, If the bag is subjected to pressure or stabbing during use, there is a risk that the pouch non-return valve closes the flow of incoming urine.

P-BAG should be used at maximum volume and for a maximum of 24 hours.

The bag is for single use only.

The printed scale is for practical guidance only and should not be used for medical decisions or clinical calculations.



Swedish Hospital Supply AB



The P-BAG can be used in a standing, sitting or lying position



- 1 - Place the bag on a smooth surface
 - 2 - Press with your hand on the center of the bag.
 - 3 - Open the non-return valve by carefully inserting e.g. an ear-tip or the tip of a pen in the hole of the bag. You should notice that air is being released out of the bag.
- Scan the code or visit our website for instructions.



Secure-Innovative-Sympathetic