



Swedish Hospital Supply AB

UNI-BAG



Always make sure that the handle of the funnel points towards the top edge of the bag before using/delivering it to the patient.

Make sure the bag is not subjected to pressure during use. If the bag is subjected to pressure during use, the check valve of the bag may shut off the flow of incoming urine.

UNI-BAG should be used at maximum volume and for a maximum of 24 hours. It is for single use only. The printed scale is for practical guidance only and should not be used for medical decisions or clinical calculations.



UNI-BAG can be used both lying, sitting or standing position



Tear in the "V" notch on the top edge of the bag or cut up with scissors to empty the UNI-BAG after use.



Swedish Hospital Supply AB

Safe-Innovative-Compassionate



In the case of air in the bag before use

- 1 - Place the bag on a smooth surface
 - 2 - Press with your hand on the center of the bag.
 - 3 - Open the check valve by carefully inserting, for example, a Qtip or the top of a pencil into the hole on the bag. You should note that air is released from the bag.
- Scan the code or visit our website for instructions.



Safe-Innovative-Compassionate