



## Swedish Hospital Supply AB

### UNI-BAG Datasheet

Issuer:	Date:	Edition:	BASIC UDI-DI:	Designation:
SHS, Swedish Hospital Supply, AB	2025-12-18.	3.	B-17350026250188	UNI-BAG

### Product Description

Urine collection bag with mounted funnel (UNISEX).

### Intended Use

**The urine collection bag, UNI-BAG** is an exceptionally good alternative to other urine collecting aids in e.g. hospital wards, home care and elderly care. Uni-BAG comes with a mounted UNISEX funnel and is used in most situations and positions (lying down, sitting, standing).

UNI-BAG should be used at maximum volume and for a maximum of 24 hours. It is for single use only. The printed scale is for practical guidance only and should not be used for medical decisions or clinical calculations.

### Features and benefits

- **UNI-BAG** is a closed-environment urine collection bag that can replace other collection vessels and is suitable for both women and men in most situations (lying down, sitting, standing).
- **The UNI-BAG** comes with a mounted funnel and carrying handle that makes it easy to grip and hang up. UNI-BAG can be used several times a day thanks to a unique check valve.
- **UNI-BAG** makes it easier for staff. The use of UNI-BAG means that staff have no direct contact with urine and odors are retained in the bag.
- **UNI-BAG** saves time for staff and money because in most situations it can be used several times in a day. Maximum volume approximately 1 500 ml or use max 24 h.
- **UNI-BAG** is made in Sweden from the material Polyethylene.



## Swedish Hospital Supply AB

### Bag

Material: Polyethylene foil, LDPE, transparent, green print

Length: Outer edge Weld. 320mm + 5mm

Width: Outer edge Weld. 183mm + 5mm

Thickness: 0.10mm + 0.01mm

Print: According to the picture or attached sample.

Printing color: Green

### Check valve

Material: Speedflex EMS No. 3-05. According to the attached sample.

Punched in – other foil printed part – of the bag that interacts with injection-moulded funnel receiver, welded to the bag.

### Funnel receiver

Material: Injection moulded polyethylene see picture or attached sample.

### Funnel

Material: Injection moulded polyethylene see attached sample.

Handle direction: Recommended, straight up. Adjustable for patient fit.

Article No	UDI-DI-Cardboard	Pcs/carton	Weight/carton
100002-A	17350026250003	50 pcs/carton	approx. 2.5 kg
	17350026250140	2000 pcs/Pallet	approx. 120 kg

### Packaging

Material: Unbleached corrugated cardboard box with inner bag made of transparent polyethylene.

Outer dimensions long: 400 mm

External dimensions width: 300 mm

External dimensions raised: 400 mm

Number of details/carton: 50 pcs.



## Swedish Hospital Supply AB

Number of cartons/pallet layer: 8 pcs.

Number of layers/pallets: 5 pcs.

Tot. Number of carton/pallet: 40 pcs.

Carton labeling: UDI-DI of the respective packaging volume, LOT, date of manufacture (year, month, day), expiration date (year, month, day).

### **Production**

Procedures: The product is produced in accordance with ISO 9001.

### **Environment**

Procedures: The product is produced in accordance with ISO 14001.

### **Country of origin**

Sweden

### **Sterile product**

No

### **Shelf life**

4 years, unbroken packaging

### **Inventory and storage**

Store dry at room temperature.

Avoid exposure to high temperatures above 55 C

### **Waste management**

According to local regulations. Plastic recycling or incineration.

### **Other information**

HS Code/Commodity code: 39269097

<https://swehospitalsupply.se/en/>



Swedish Hospital Supply AB



Secure-Innovative-Sympathetic