



Swedish Hospital Supply AB

Questions and Answers, P-BAG SAP:

Question: Within home healthcare in our municipality, we have for some time regularly used the P-BAG SAP which contains superabsorbent. How do we ensure that it will not leak when we connect the funnel to the bag?

Answer: Press the funnel into the bag's sleeve until you hear a click; this means that the funnel is correctly connected to P-BAG SAP.

Question: Once we have connected the funnel to the bag, in which direction must the funnel point when it is being used?

Answer: The handle of the funnel must point towards the top of the bag.

Question: Can our female patients use the P-BAG SAP?

Answer: Yes, the P-BAG SAP is unisex and certainly can be used by women and men while lying down, sitting and standing. The patient holds the handle and the lower / opposite section of the funnel connects tightly but not hard between the anus and vagina whilst being used.

Question: We have largely succeeded in replacing "adult nappies" for our users by giving them the P-BAG SAP instead. This has essentially led to an improvement in the quality of life and integrity of these users at the same time as our municipality saves money since nappy use per day and user is basically much more expensive. Now to our question: what is the best way to destroy the P-BAG SAP?

Answer: The P-BAG SAP contains superabsorbent - even if the bag is full, its superabsorbent capacity is still active. This means that the bag's content must not be emptied down the toilet or into something similar. You must destroy the bag as per your procedures for the destruction of nappies. The difference between the destruction of a nappy and a P-BAG SAP is that the content of the bag is sealed in and odour-free because the bag has a check valve which means that it can be discarded in with ordinary household refuse.

Question: We have noted that air can get into the bag before use and that, in some cases, this prevents urine from entering the bag. What do we do about this?

Answer: The bag is equipped with a check valve to prevent leaks and bad odour. Air in the bag is easily removed by laying the bag on a firm surface and pushing a pen or similar into the hole in the bottom of the funnel. Press the bag with your hand and this will evacuate the air in the bag.

Address
Gårdfarivägen 3
53238 SKARA
Sweden

Phone
+46703491203
Email
info@shsab.eu

Plusgiro: 980054-1
Bankgiro: 5484-9112

Corporate ID
559251-3534
VAT reg. no
SE559251353401