



Swedish Hospital Supply AB

NOSEEEZZ®

Instruction for use

1) Place the Noseezz tip between your middle finger and index finger. Then place your thumb underneath the Noseezz. Same grip as on a container of nasal spray.



2) Then press the Noseezz together. Keep the grip and Noseezz compressed as in picture 2.



3) Place the Noseezz top carefully against the child's nostril so that it is tight between the nostril and the top.



4) When the tip is placed against the nostril and there is a tight fit between the tip and the child's nostril. Gently release the Noseezz "bellows" to suck and clean the nose from snot.



Secure-Innovative-Sympathetic



Swedish Hospital Supply AB

NOSEEZZ®

Recommended cleaning: gently soak up boiling or hot water with Noseezz. Shake the Noseezz and forcefully push the water out. Clean the Noseezz regularly after use.



www.swehospitalsupply.se

Secure-Innovative-Sympathetic